## FEATURES

1. **LOWEST VERTICAL HEIGHT:** The total height of the XXX Bar Attachment (abutment plus male) is only 2.5mm on a cast alloy bar.
2. **LOCATING DESIGN:** Self-locating design allows a patient to easily seat their overdenture without the need for accurate alignment of the attachment components.
3. **RETENTION INSIDE AND OUT:** The patented *Dual Retention* innovation provides the XXX Attachment with greater retention surface area than ever before available with other attachments. A combination of inside and outside retention ensures the longest lasting performance.
4. **ROTATIONAL PIVOTING ACTION:** The design of the pivoting XXX Male allows a resilient connection for the prosthesis without any resulting loss of retention. The retentive nylon male remains completely in contact with the abutment socket while its metal denture cap has a full range of rotational movement over the male.
5. **CHOICE OF FOUR TECHNIQUES:** The XXX Bar Attachment can be placed by any of the four popular techniques used by Dental Laboratories to fabricate an implant retained bar with attachments:
  1. Cast a Castable Threaded Insert into a bar for a removable threaded XXX Bar Female connection.
  2. Drill and tap the bar for a threaded XXX Bar Female.
  3. Laser weld a XXX Laser Bar Female to the bar.

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4. Cast-to the stainless steel XXX Cast-  
To Bar Female with gold alloy.

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